

Surface Disinfection

Using CaviCide: Spray ▶ Wipe ▶ Spray

Clean before disinfecting



1. Spray CaviCide directly onto surface to be cleaned.



2. Wipe with clean paper towel to remove debris and bioburden.



3. Discard towel.

Use CaviCide® to clean and disinfect:

Potentially contaminated noncritical instruments and equipment

Hard, nonporous surfaces or items to prevent cross contamination, such as:

- Light fixtures
- Clinical and laboratory surfaces
- Telephone and office machines

Note: Follow standard precautions for personal protection as appropriate.

Disinfect after cleaning



4. Spray CaviCide directly onto pre-cleaned surface, thoroughly wetting area. Follow label instructions for appropriate contact times.



5. Wipe with clean paper towel.



6. Discard towel.

U.S. PART NO.	CANADIAN PART NO.	PRODUCT DESCRIPTION	PACKAGING
13-1002	N/A*	CaviCide – 2 ounce spray	48 bottles/case
13-1008	N/A*	CaviCide – 8 ounce spray	12 bottles/case
13-1024	11-1024	CaviCide – 24 ounce spray	12 bottles/case
13-1000	11-1000	CaviCide – 1 gallon	4 bottles/case
13-1025	11-1025	CaviCide – 2.5 gallon	2 bottles/case

*Note: CaviCide 2 ounce and 8 ounce are not sold in Canada.

